

Edan F6 FHM Quick Start Guide

- Power toggle switch is on the right side, press and hold briefly.
- Plug the fetal US transducer into the left side of the monitor US1 blue port, then using gel, find FHT's. Once found, secure the transducer with a belt or belly band.
 - The FHT tracing will be in the upper portion of the screen and the numerical will report under US1.
- Place the toco transducer on the fundus of the uterus and secure with a belt or belly band. Plug the green toco into the port on the left side of the monitor. Press "auto zero" button.
 - Contractions read out on the lower screen. Remember this gives you the duration and length between contractions but not the strength.
- Press "start" button to begin monitoring.
 - This should prompt you to enter ID (flight number and pt name).
 - This will briefly clear the screen, auto zero, and begin printing.

Press "print" button to stop paper recording. The monitor will continue to display FHT.

Main setup key is the right upper corner.  Use this to adjust alarms, and volume.



Paper advance key to advance paper.



1

2

3

1- Searching Key- Use this to search for past cases.

2- 3 Use these keys to toggle through case. Once back at beginning of case you may print entire strip.

- **Twin monitoring-** Use the second US transducer and plug it into the blue US2 monitor port. The recording will be in two different colors. They may overlap and make it difficult to monitor. To offset US2 tracing: use the dial to highlight and select the "setup" → "recorder" → FHR2 "offset" and choose either -20, which will show up below FHR1, or +20 and it will show up above FHR1.